Actreen® Hi-Lite Cath

EN Instructions for use
DE Gebrauchsinformation
ES Instrucciones de uso
FR Mode d'emploi
IT Istruzioni per l'uso
NL Gebruiksaanwijzing
AR تعليمات الاستخدام
BG Инструкции за употреба

CA Mode d'emploi
CS Návod k použití
DA Brugsanvisning
EL Οδηγίες χρήσης
FI Käyttöohjeet
HU Használati utasítás
LV Lietošanas norādījumi
MS Arahan penggunaan

NO Bruksanvisning
PL Instrukcja użycia
PT Instruções de utilização
RO Instrucțiuni de utilizare
RU Инструкция по применению
SK Návod na použitie
SV Bruksanvisning
TR Kullanma kılavuzu

B | BRAUN

EN INSTRUCTIONS FOR USE

Actreen® Hi-Lite^{Cath} Intermittent Urinary Catheters are indicated for intermittent urinary catheterisation by patients with chronic urine retention or voiding dysfunction.

Actreen® Hi-Lite^{Cath} is a sterile, pre-lubricated, ready-to-use catheter.

1. PREPARATION:

- Carefully wash hands before and after catheterisation, and clean the area surrounding the urethral opening.
- Before opening**, check the product is intact and make sure the lubricant is evenly spread over the catheter by touching the catheter over the packaging.

2. PRODUCT OPENING:

For safety and hygienic reasons, we advise you to use the product in shortest possible time after opening.

Remove the protective film on the connector side:

Grip and tear off the protective film at mark «1».

Remove the protective film on the catheter side:

FOR THE FOLDABLE NO TOUCH TECHNIQUE






- Option 1: Grip and tear off the protective film at mark «A».
- Option 2: Hold the catheter in one hand and with the other hand insert your thumb in the eyelet at mark «A» to remove the protective film*.
- Then fold down the sheath into accordion folds without touching the catheter.

FOR A SIMPLIFIED NO TOUCH TECHNIQUE

- Release a movable no touch sheath by gripping and tearing off the film at mark «B». This movable part will protect the catheter when handling.

IF YOU WANT TO RELEASE THE WHOLE CATHETER:


- After removing the film at mark «1» from the connector side, hold the cone and lift the catheter out of the packaging.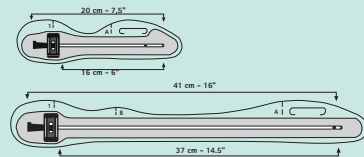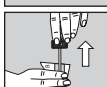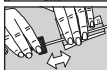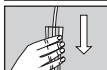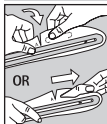

3. CATHETERISATION:


Actreen® Hi-Lite^{Cath} can be connected to a urine collection bag.

MALE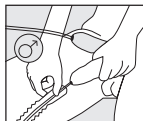

- With one hand hold the penis upwards to reduce curvature of the urethra.
- With the other hand carefully introduce the catheter into the urethra until it reaches the bladder. Point your penis downwards in case of resistance. Continue inserting the catheter until urine starts to flow.
- Tiemann/Coudé catheter : In order to correctly guide the catheter, ensure that the mark on the connector is pointing upwards.

FEMALE

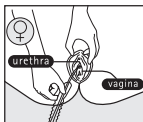- With one hand spread the outer lips of the vulva to locate the urethral opening. With the other hand carefully introduce the catheter into the urethra until it reaches the bladder and the urine drains.

MALE / FEMALE

- Manual pressure on the lower abdomen towards the end of catheterisation will ensure that the bladder has been completely emptied.
- When the bladder is completely empty, gently withdraw the catheter.
- To stop the catheter dripping you can turn up the connector end of the catheter when removing.
- Then discard the catheter according to the current local regulations.

Self-catheterisation should only be carried out under medical advice and only in accordance with the instructions given.

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B. Braun Medical A/S
B. Braun Medical AB
B. Braun Medical Inc.
B. Braun Medical SAS | 204 Avenue du Maréchal Juin | 92100 Boulogne-Billancourt | France